

Short operating instructions

Indication of operation

Indicator for operation, warnings, errors

Infrared interface

Normal programme


Cleaning and disinfection of soiled care utensils.

Rim flushing

(for separate slop sink)
Option wash chamber rinsing

Display

for warnings or errors
contact technical service.
Acknowledge key
acknowledges errors
switches off the signal.

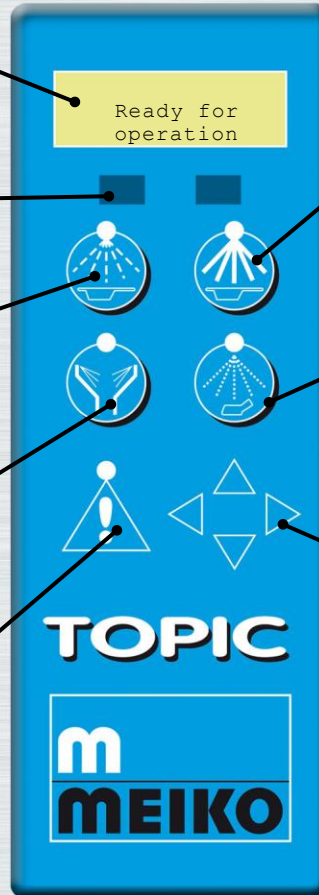

Intensive programme

Cleaning and disinfection of heavily soiled care utensils.

Short programme


Cleaning and disinfection of soiled care utensils. (e.g. urine bottles)

Cursor key pad

▲ ▼ keys for number of programme sequences
◀ ▶ keys see detailed operating instructions



Open wash chamber door
Infrared or optional pedal switch.

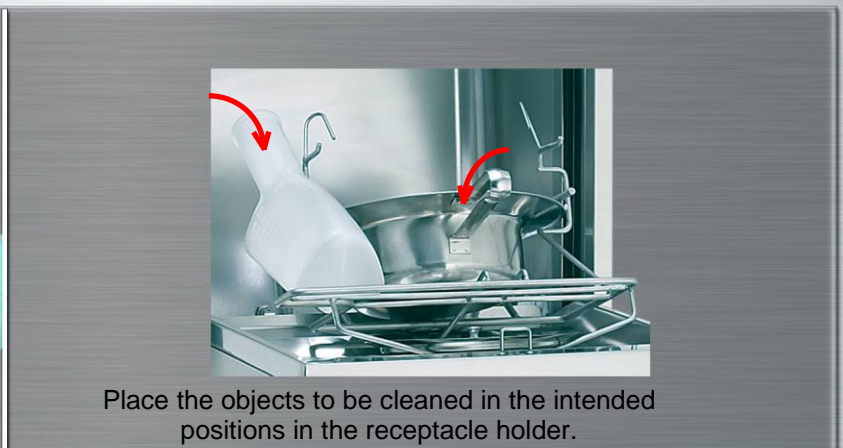

Place the objects to be cleaned in the intended positions in the receptacle holder.



Select the cleaning programme to reflect the degree of soiling and start the programme by lightly pressing the corresponding programme key. The door closes automatically.



At end of programme
open washing chamber door.
Infrared or optional pedal switch.



Crushing hazard

Door movement can crush fingers, hands and arms.

- Do not reach into the machine when the door is in motion.
- Contact the home automation or service department to remove any door blockages.